

Fitting & Care Instructions

Model: DCT-WTFLEX



Instructions on putting on brace:



Step 1: Unstrap brace and slide injured hand into sleeve with the palm down.



Step 2: Move large Velcro platform for tighter fit if needed. Tighten and adhere top and bottom strap to desired support



Step 3: Wrap inner thumb strap around thumb and adhere to opposite side.



Step 4: Wrap outer thumb strap and adhere to opposite side



Step 5: Tighten thumb strap around thumb at desired support



Step 6: Locate the key hole on the side of the thumb. Place provided allen wrench in the key hole and turn to adjust the extension and flexion of the thumb. Set at desired flexion/extension

WARRANTY

The DCT-WTFLEX is covered by a 6 month warranty against all mechanical or manufacturer defects, from the date of purchase.

CARE INSTRUCTIONS

Hand wash with mild detergent and air dry.